



Product Main Number 1000027524



Shaped pad, ABENA San Basic, 9, Green colour code

- ✓ Textile-like back sheet: Comfortable and breathable
- ✓ Leakage barriers provide extra protection
- ✓ Effective core technology for fast absorption
- ✓ Anatomically shaped for a better fit
- ✓ Wetness indicator
- ✓ Odour Control system minimises unpleasant smells.



Product Description

Abena San Basic is our range of anatomically shaped pads with textile-like, breathable backsheet that ensures discreet protection and comfort for moderate to heavy incontinence. The anatomically shaped pads have an excellent fit that allows free movement. The effective core provides fast absorption and a dry surface, even after repeated urination. All products are fitted with leakage barriers for extra protection and the Odour Control system minimises unpleasant odours. The wetness indicator turns blue when the product is wet, indicating the right time to change.



Specifications

Base name	Shaped pad
Brand	ABENA
Sub-Brand	San Basic
Size	9
Colour	White
Features	Green colour code
Material	ECF fluff, PE, breathable textile backsheet (BTBS), elastane, nonwoven, PP, resin, SAP
Length/depth	73 cm
Width	37 cm
Absorption Rothwell	2200 ml
Certificates	CE. MD. UKCA.
CE Class (Medical Devices)	Class I
Shelf Life	5 years

Instructions for use/application

Anatomically shaped pads for moderate to heavy incontinence. ABENA San Basic products fit almost all users, as body shape does not have to be taken into account, since the product only fills between the legs and not around the stomach and hips. The range features a numbered and colour-coded system to make it easy to distinguish between different sizes and absorbency levels. ABENA San Basic should always be used with special fixation underwear, such as our ABENA Fix products. The fixation pants ensure that the pad is kept close to the body and reduces the risk of leakage. Pull the fixation pants up to the middle of your legs. Fold the pad lengthwise to create a bowl shape with the back facing outwards. Pass the pad between your legs from front to back. Place the top edge of the pad at the tailbone. Fold out the back of the pad horizontally to create a bowl shape between your legs. Use the wet indicator to check if the product is positioned exactly in the centre. Fold the pad out horizontally in front. Pull the fixation pants up into place.

Packaging data

Unit	Contains	Length	Width	Height	EAN
cll	80 pcs	60 cm	40 cm	45.2 cm	5703538509322
pck	20 pcs	32.5 cm	20 cm	29.5 cm	5703538509315
pcs	1 pcs				

MD Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.

CE The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products and medical equipment.

UK CA The UKCA mark guarantees that a product is safe to use and complies with all safety precautions in line with product standards and legislation of the UK.

ODOR CONTROL Odor control is a critical factor in ensuring discreteness for the user. ABENA's incontinence products are designed and engineered to ensure that minimal Odor arises from the product.

PROTECTIVE LEAKAGE BARRIERS Effective leakage barriers, both visible and invisible and latex-free elastics ensure a snug fit, which reduces the risk of leakages.

WETNESS INDICATOR Wetness indicator is a liquid-sensitive strip that changes colour or fades out indicating when it is time to change the product.

LATEX The product does not contain latex.

