



Product Main Number 1000031814

ABENA®

Kinesiology tape, ABENA, 5m x 5cm, Pack of 2, black and blue, latex-free, non-sterile

- ✓ Kinesiology tape made of cotton and spandex ensures comfort and durability
- ✓ 2 rolls per pack in the colours blue and black
- ✓ Latex-free material minimises the risk of allergic reactions
- ✓ Elastic fibres support optimal blood flow and muscle relaxation



Product Description

Abena kinesiology tape is made from a blend of cotton and spandex, ensuring optimal elasticity and comfort. The elastic fibre structure improves blood flow and helps to reduce muscle tension, as well as supporting the activation and offloading of muscles. This makes the tape ideal for recovery and prevention of sports injuries, as well as for support of tendons and joints. The tape is latex-free.



Specifications

Base name	Kinesiology tape
Brand	ABENA
Features	Pack of 2, black and blue, latex-free, non-sterile
Single or multiple use	Single use
Material	Cotton, spandex
Length/depth	5 m
Width	5 cm
Certificates	CE. MD.
CE Class (Medical Devices)	Class I
Directives, regulations and acts	MDR (EU) 2017/745
Safety Instructions And Warnings	Consult a doctor or physiotherapist if you have injuries or are unsure. People with sensitive skin should test the product on a small area of skin for 24 hours before use. If irritation occurs, you should stop use immediately. Do not apply the product directly to a wound. For external use only.
Shelf Life	According to expiry date
Storage Instructions	Store dry, clean and at room temperature.
Product Disposal Instructions	Can be disposed of with normal household waste sorted according to local regulations.
Packaging Disposal Instructions	Can be disposed of with normal household waste sorted according to local regulations.

Instructions for use/application

Apply the tape directly to the skin. The best results are achieved if the skin is clean and dry. The tape can be used for up to 7 days, but it is recommended to change every 3 days to achieve the best possible effect. Cut the tape into suitable lengths, then round the corners. Apply the tape as instructed according to your needs.

Packaging data

Unit	Contains	Length	Width	Height	EAN
cll	112 rl	32 cm	43 cm	33 cm	6978479040275
pck	2 rl	5.8 cm	7.5 cm	15.5 cm	6978479040268
rl	1 rl				



Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.



The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products and medical equipment.



The product is only for single-use.



The product does not contain latex.

