



Product Main Number 1999911959




Bodystocking, ABENA Fix Bodystocking, XL, white, with zips in the leg, crotch and shoulder areas

- ✓ A cotton-based soft, breathable bodystocking for fixation of a incontinence product
- ✓ Zips in the leg, crotch and shoulder allow easy application and adjustment
- ✓ Single packed"



Product Description

Textile fixation pant for fixing a pad, worn as full-body underwear. The ABENA Fix Bodystocking is a soft and flexible bodystocking made of cotton and with sleeves and legs. The cotton material allows moisture to leave the skin, so the user feels dry and comfortable at all times. Discreet zips are situated in the leg, crotch and shoulder area allowing easy application and adjustment. The zipper is covered, protecting the skin from irritation. Size and washing instructions are indicated on the garment label. ABENA Fix Bodystocking has a colour code on its collar for easy product identification. And the packaging and product also contain information about product name, size, washing instructions and material composition. Made of 75% cotton, 20% polyamide and 5% elastane.

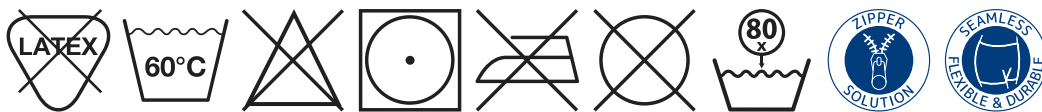

Specifications

Base name	Bodystocking
Brand	ABENA
Sub-Brand	Fix Bodystocking
Size	XL
Colour	White
Features	With zips in the leg, crotch and shoulder areas
Material	Cotton, elastane, PA
Ingredients / Composition	75% cotton, 20% polyamide, 5% elastane
Hip size	95-120 cm
Gender	Unisex
Certificates	CE. OEKO-TEX. MD. UKCA.
CE Class (Medical Devices)	Class I
Directives, regulations and acts	MDR (EU) 2017/745
Shelf Life	5 years
Storage Instructions	Store dry, room temperature and no direct sunlight.
Washing Instructions	Can be washed at up to 90 °C, but best durability achieved at 60 °C. Tumble dry on gentle program. Washable approx. 80 times. Close zipper before washing and place zipper head in position where it is fully covered by the cover at the end of the zipper. 8% shrinkage. The products should be washed in a laundry bag, which helps to extend shelf life. Use of bleaching agents may compromise the shelf life of the product.
Product Disposal Instructions	Can be disposed of with normal household waste sorted according to local regulations.
Packaging Disposal Instructions	Can be recycled or incinerated.

Instructions for use/application

Textile bodystocking for fixing a pad, worn as full-body underwear. It is important to select correct hip size for comfortable wear. Best durability achieved at

60° C washing. Close the zip before washing and place the zip head in a position where it is fully covered by the cover at the end of the zip. Do not expose the product to pressure such as pressing or calendering as this may damage the zips.

Packaging data

Unit	Contains	Length	Width	Height	EAN
cll	12 pcs	40 cm	29.8 cm	25.4 cm	5703538475504
pcs	1 pcs				5703538475498



Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.



The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products and medical equipment.



The UKCA mark guarantees that a product is safe to use and complies with all safety precautions in line with product standards and legislation of the UK.



OEKO-TEX, also known as "Confidence in Textiles", is a health and environmental label used throughout the world. The certificate ensures that textiles do not contain substances that are harmful to health or the environment.



The product does not contain latex.



The product can be washed at 60 degrees or below.



Do not bleach the product.



The product can be dried in the tumble dryer at low temperatures.



Do not iron the product.



Do not dry clean the product.



The product can be washed up to 80 times.



Easy and secure zip closure instead of buttons. Designed for quick access and improved comfort.



Seamless design for extra comfort and flexibility. No side seams means less irritation and more durability.